

REV.	DATE	REVISION HISTORY	Approved By
1.0.0	-	INITIAL RELEASE	NILESH
1.0.1	11 SEP 17	UPDATED ACCORDING 3.2 VER.	NILESH
1.0.2	31 MAR 19	UPDATE AGAIN ECR - J-QA-ECR-2019-0027 Change GSM & Not Required Lamination	Dilkash M.
1.0.3	30 SEP 19	Updated Against CAPA-R&D-17-003	Harshit P.
1.0.4	12 DEC 22	UPDATED AGAINST J-QA-ECR-2022-17 ADDED QR CODE	Dilkash M.

QUICK REFERENCE GUIDE

HOW TO OPERATE WINSTIM:

STEP 1: CONNECT ADAPTOR, ULTRASOUND APPLICATOR AND ELECTRODES LEAD WIRE
STEP 2: SWITCH ON DEVICE, POWER SWITCH PROVIDED AT THE LEFT BOTTOM SIDE

The device will display Home screen with Programs and Options, icons as shown in image 1. Tap on Electrotherapy to start the treatment procedure.

STEP 3: SELECT ELECTROTHERAPY
Tap on any electrotherapy from list displayed.

The device will display Home screen with Programs and Options, icons as shown in image 1. Tap on Electrotherapy to start the treatment procedure.

STEP 4: SELECT 2/4 POLE APPLICATION
Tap on 2 or 4 pole application from list displayed.

STEP 5: SET THE CHANNEL
After selection of Electrotherapy, Treatment parameters will be shown on the screen. Select Channel by tapping its corresponding icon.

STEP 6: SET THE PARAMETERS
After channel selection tap on parameter to set the time, frequency, duty cycle and intensity.

STEP 7: START/STOP THE TREATMENT
Tap on the Start button to start the treatment, and during the process intensity can be adjusted by rotating the knob. To stop press stop button.

1 Power Switch
2 8" Touch Screen Display
3 1 & 3 Mhz Ultrasound Applicator
4 Time & Intensity Controller
5 USB Socket for Firmware Update
6 Power Input Socket
7 Output Socket Channel 1 - 2
8 Output Socket Channel 3 - 4

Register your product through this link:
www.joharidigital.com/productregistration
 Important: Warranty is not valid if you do not activate your warranty within 15 days of purchase.

Front

QUICK REFERENCE GUIDE

HOW TO OPERATE WINSTIM:

LIBRARY & ELECTRODE PLACEMENT

HOW TO OPEN WAVEFORM DESCRIPTION LIBRARY AND ELECTRODE PLACEMENT ?

STEP 1: CONNECT ADAPTOR, ULTRASOUND APPLICATOR AND ELECTRODES LEAD WIRE, SWITCH ON DEVICE, POWER SWITCH PROVIDED AT THE LEFT BOTTOM SIDE
 The device will display Home screen with Programs and Options, icons as shown in image. Tap on Electrotherapy icon to see library & electrode placement

STEP 2: SELECT ELECTROTHERAPY WAVEFORM
Tap on any waveform from list displayed.

STEP 3: OPEN LIBRARY
Tap on Library icon to open library & information

STEP 4: LIBRARY & ELECTRODE PLACEMENT GUIDE
Read description. Tap on picture to enlarge.

HOW TO OPEN PRESET TREATMENT PROTOCOLS AND ELECTRODE PLACEMENT LIBRARY ?

STEP 1: CONNECT ADAPTOR, ULTRASOUND APPLICATOR AND ELECTRODES LEAD WIRE, SWITCH ON DEVICE, POWER SWITCH PROVIDED AT THE LEFT BOTTOM SIDE
 The device will display Home screen with Programs and Options, icons as shown in image. Tap on Library icon to see library & electrode placement.

STEP 2: SELECT TREATMENTS
Tap on Treatments option from list displayed.

STEP 3: OPEN LIBRARY
Tap on any treatment to open library & information

STEP 4: LIBRARY & ELECTRODE PLACEMENT GUIDE
Read description. Tap on picture to enlarge.

Back

SPECIFICATION :

- 1. MATERIAL : Paper
- 2. PRINT/ADHESIVE : Digital
- 3. LENGTH : 297mm
- 4. HEIGHT : 210mm
- 5. THICKNESS : 250
- 6. Lamination : -

7. GENERAL TOLERANCES AS PER ISO 2768-MK.

SCALE : 1 : 2.5				NAME	DATE	SIGN
GENERAL DIMENSIONAL TOLERANCES				Designed	GR Chouhan	12 DEC 22
DEC.	DEC.	DEC.	DEC.	Checked	Sourabh S.	12 DEC 22
X +/-	.X +/-	.XX +/-	.XXX +/-	Approved	Dilkash M.	12 DEC 22
[7]	[7]	[7]	[7]			
DEC.	HOLES	FRAC.	ANG.			
.XXXX +/-	+/-	+/-	+/-			

DESCRIPTION : QRF WinStim WSAU		DRAWING NAME / NO.	52035012	A4	
		JDHL PART No.	30CFG00054		NO. 01
		REV. :	1.0.4		OF. 03

QUICK REFERENCE GUIDE


HOW TO OPERATE WINSTIM:

STEP 1: CONNECT ADAPTOR, ULTRASOUND APPLICATOR AND ELECTRODES LEAD WIRE

STEP 2: SWITCH ON DEVICE, POWER SWITCH PROVIDED AT THE LEFT BOTTOM SIDE


The device will display Home screen with Programs and Options, Icons as shown in image 1. Tap on Electrotherapy to start the treatment procedure.

STEP 3: SELECT ELECTROTHERAPY
Tap on any electrotherapy from list displayed.

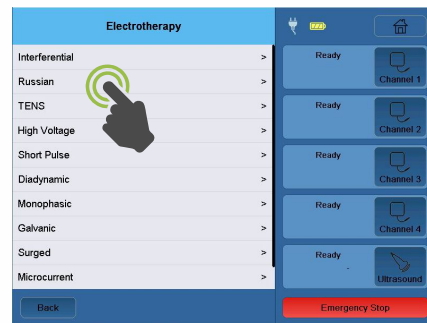




STEP 4: SELECT 2/4 POLE APPLICATION
Tap on 2 or 4 pole application from list displayed.

STEP 5: SET THE CHANNEL
After selection of Electrotherapy. Treatment parameters will be shown on the screen. Select Channel by tapping its corresponding icon.

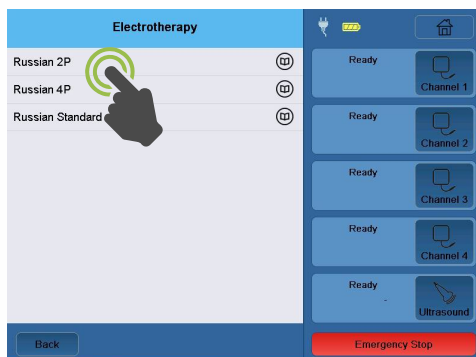

STEP 6: SET THE PARAMETERS
After channel selection tap on parameter to set the time, frequency, duty cycle and intensity.

STEP 7: START/STOP THE TREATMENT
Tap on the Start button to start the treatment, and during the process intensity can be adjusted by rotating the knob. To stop press stop button.

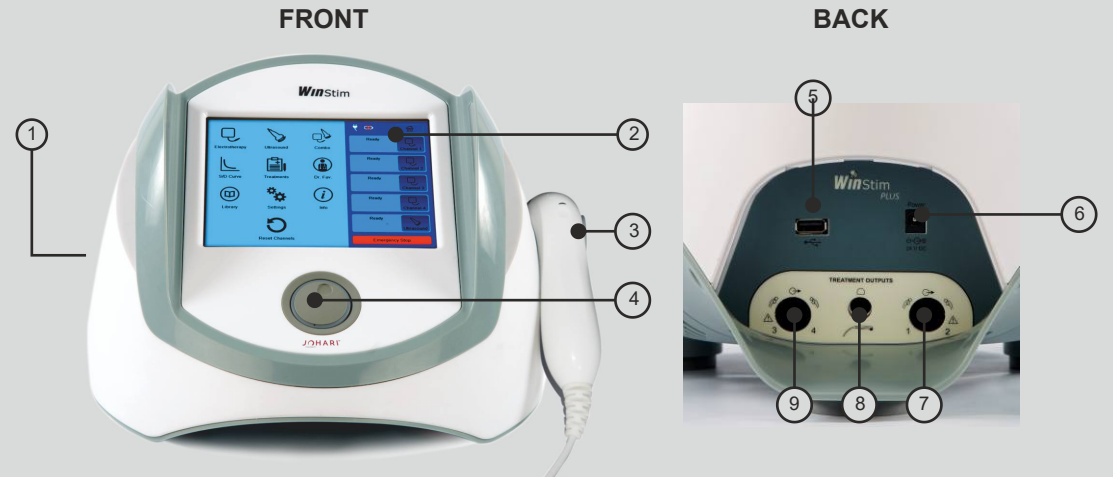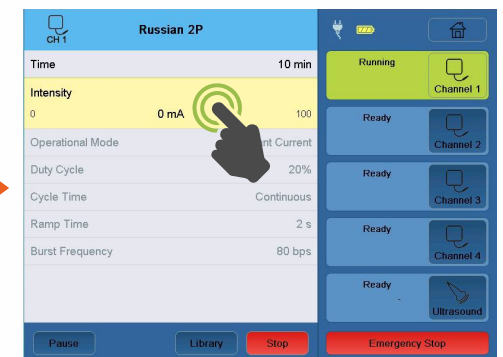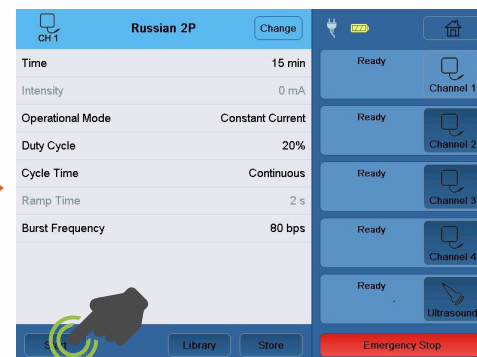

- 1 Power Switch
- 2 8" Touch Screen Display
- 3 1 & 3 MHz Ultrasound Applicator
- 4 Time & Intensity Controller
- 5 USB Socket for Firmware Update
- 6 Power Input Socket
- 7 Output Socket Channel 1 - 2
- 8 Output Socket Ultrasound output
- 9 Output Socket Channel 3 - 4

Register your product through this link:
www.joharidigital.com/productregistration


Important: Warranty is not valid if you do not activate your warranty within 15 days of purchase.



HOW TO OPEN WAVEFORM DESCRIPTION LIBRARY AND ELECTRODE PLACEMENT ?


STEP 1: CONNECT ADAPTOR, ULTRASOUND APPLICATOR AND ELECTRODES LEAD WIRE, SWITCH ON DEVICE, POWER SWITCH PROVIDED AT THE LEFT BOTTOM SIDE

The device will display Home screen with Programs and Options, Icons as shown in image. Tap on Electrotherapy icon to see library & electrode placement.

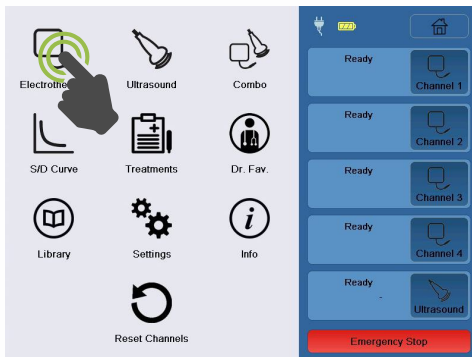

STEP 2: SELECT ELECTRO-THERAPY WAVEFORM

Tap on any waveform from list displayed.

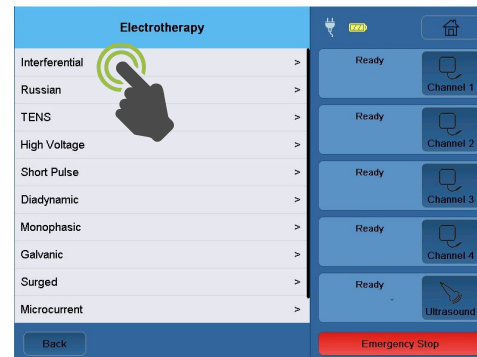


STEP 3: OPEN LIBRARY

Tap on Library icon to open library & information



STEP 4: LIBRARY & ELECTRODE PLACEMENT GUIDE


Read description. Tap on picture to enlarge.

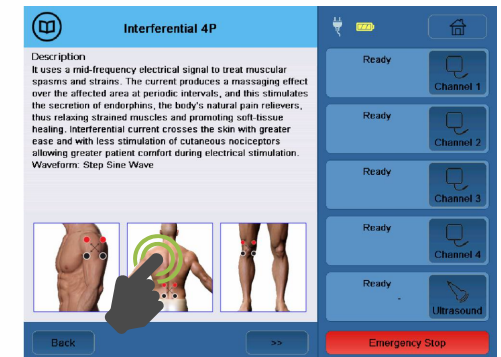

HOW TO OPEN PRESET TREATMENT PROTOCOLS AND ELECTRODE PLACEMENT LIBRARY ?


STEP 1: CONNECT ADAPTOR, ULTRASOUND APPLICATOR AND ELECTRODES LEAD WIRE, SWITCH ON DEVICE, POWER SWITCH PROVIDED AT THE LEFT BOTTOM SIDE

The device will display Home screen with Programs and Options, Icons as shown in image. Tap on Library icon to see library & electrode placement.

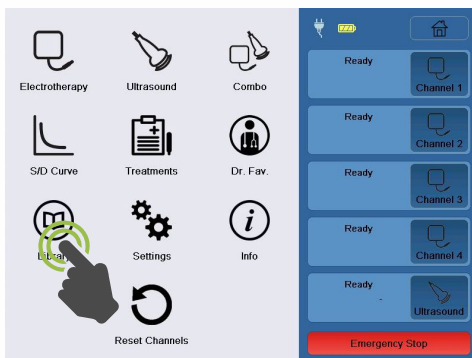

STEP 2: SELECT TREATMENTS

Tap on Treatments option from list displayed.

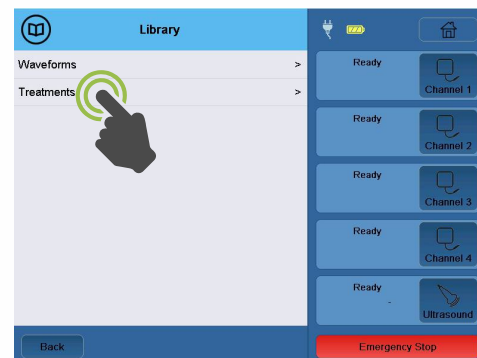

STEP 3: OPEN LIBRARY

Tap on any treatment to open library & information



STEP 4: LIBRARY & ELECTRODE PLACEMENT GUIDE

Read description. Tap on picture to enlarge.

